

# BOM i-TOUCH-S

**Product name :** Semi-permanent make-up machine

**Model name :** BOM-i-TOUCH-S



This product is 'Medical device', please be sure to read the instructions before use and follow the user manual. Please contact us at the contact number below if you have any inquiries about the product.


## Composition and Introduction

### Composition

Body + Body Cradle + Hand-piece + Hand-piece stand +  
Foot Switch + Adapter + Cable

### Introduction

It marks the color pigments inhaled into the needle by injecting them to skin by a back-and-forth motion. MTS machine helps medical supplies be absorbed to skin by irritating.

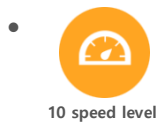




Pause on the stand/ by foot switch



Dual functions of MTS/PMU

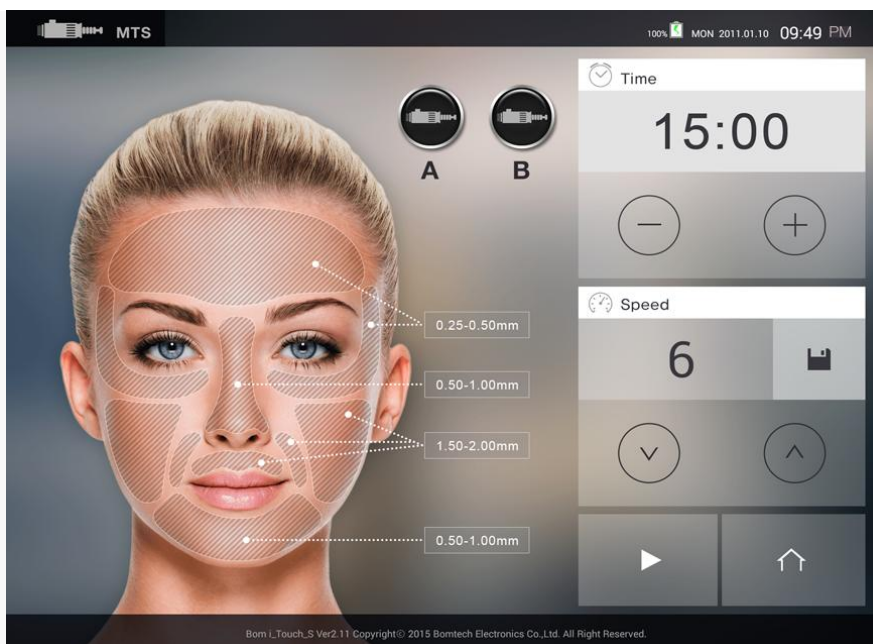



## Additional Functions

- take a photo with built-in camera and manage customers.
- Play music to provide customers with the comfortability and relaxation.(Earphone & Headphone available)
  - Video play function / Sample photos provided
  - Wide touch screen




## Easy to use

- Pause right after putting down the hand-piece on the stand.
- Power ON/OFF with foot switch.
- Free movement allows convenient treatment.
- Selectable between two hand-pieces.
- Save the current speed.



### Streamlined design of Hand-piece

- Comfortable grip feelings with streamlined design
- Selectable disposable needle cartridge between PMU and MTS for its purpose.
- Operate with various types of needle.

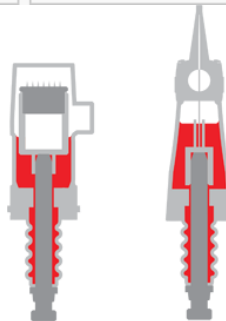


### Disposable sterilized cartridges

- EO gas Sterilized disposable needle cartridges  
(Purchase separately, consumables)



## Applied our own patent technology

- Hygienically prevents from being injected contaminant during the operation.
- The amount of pigmentation emission of DIGITAL REVO cartridges is adjustable.

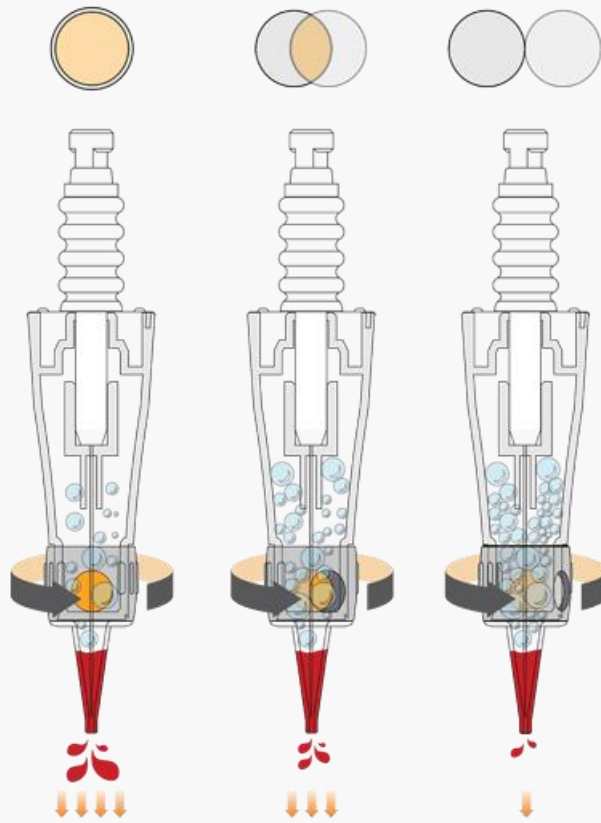

## Adjust the amount of pigment emission

- Adjust the amount of pigment emission by turning the adjustment ring.



### Strong packaging of needle cartridges

- Packaging with transparent materials
- Prevent damage to products by using strong packaging



# BOM i-TOUCH-R

**Product name** : Semi-permanent make-up machine

**Model name** : BOM-i-TOUCH-R



This product is 'Medical device', please be sure to read the instructions before use and follow the user manual. Please contact us at the contact number below if you have any inquiries about the product.

## Composition and Introduction

### Composition

Body + Hand-piece + Hand-piece stand + Foot Switch +  
Adapter + Cable + Screw

## Introduction

It marks the color pigments inhaled into the needle by injecting them to skin by a back-and-forth motion. MTS machine helps medical supplies be absorbed to skin by irritating.



10 speed level



Touch Screen





Pause on the stand/ by foot switch



Dual functions of MTS/PMU



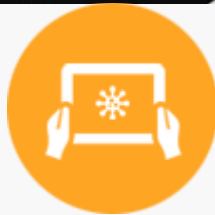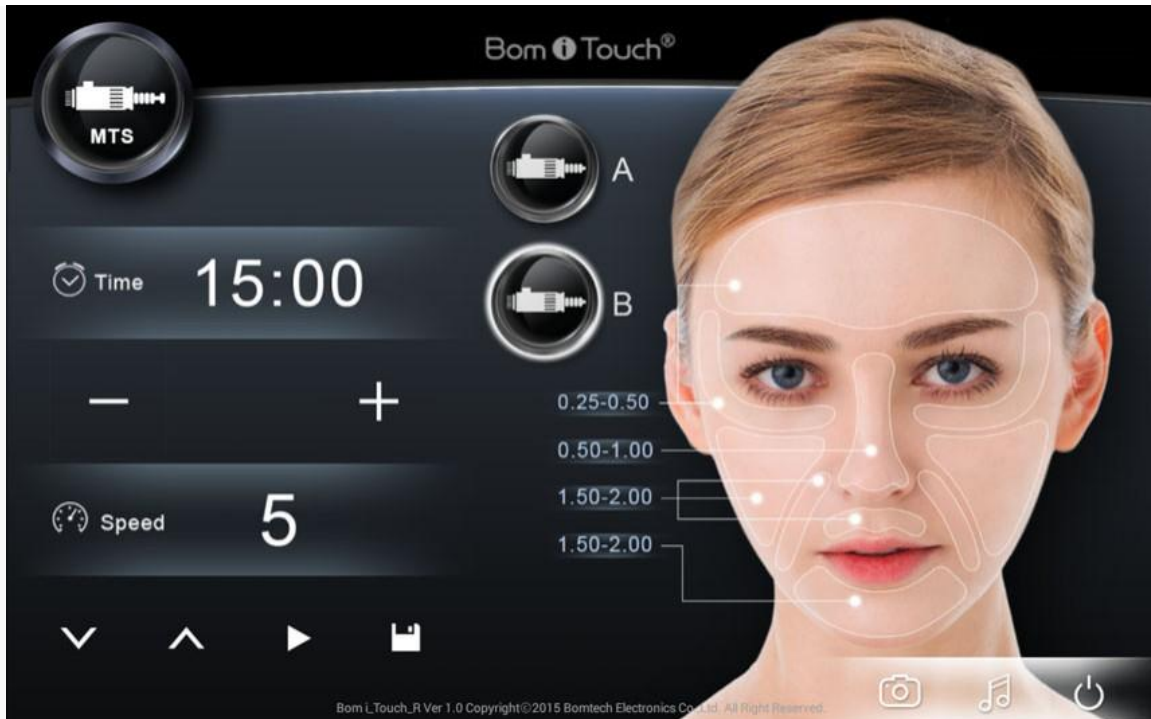

### Additional Functions

- take a photo with built-in camera and manage customers
- Play music to provide customers with the comfortability and relaxation.(Earphone & Headphone available)



## Easy to use

- Pause right after putting down the hand-piece on the stand.
- Power ON/OFF with foot switch.
- Selectable between two hand-pieces.
- Save the current speed



## Streamlined design of Hand-piece

- Comfortable grip feelings with streamlined design
- Selectable disposable needle cartridge between PMU and MTS for its purpose.
  - Operate with various types of needle.




### Disposable sterilized cartridges

- EO gas Sterilized disposable needle cartridges  
(Purchase separately, consumables)



### Applied our own patent technology

- Hygienically prevents from being injected contaminant during the operation.
- The amount of pigmentation emission of DIGITAL REVO cartridge is adjustable.

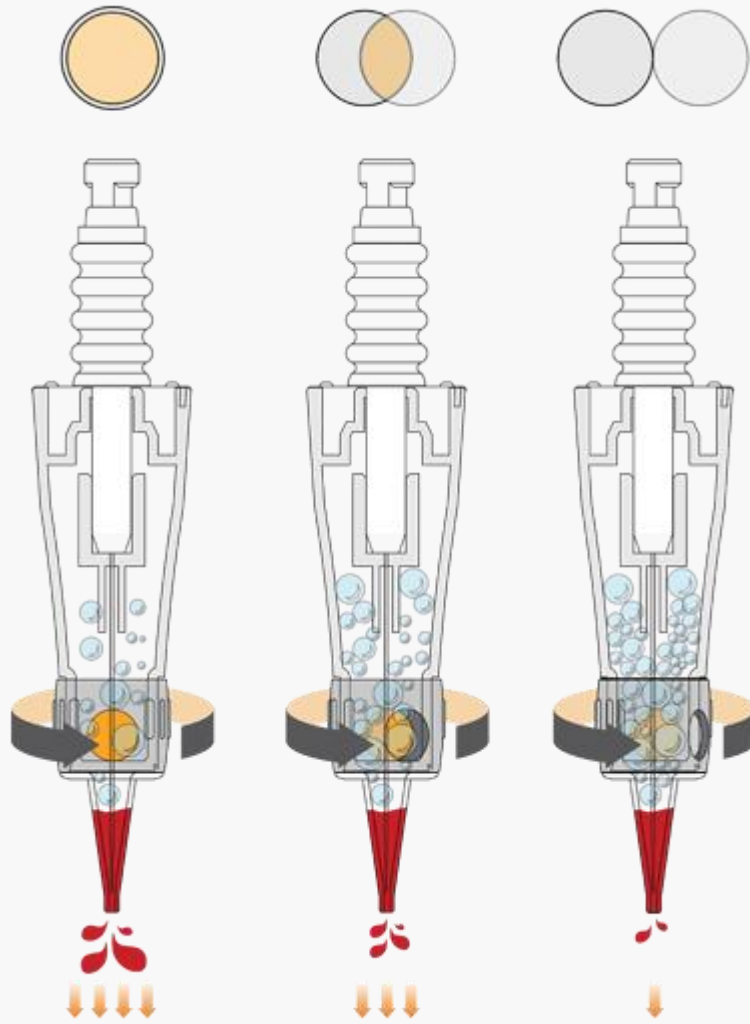

### Adjust the amount of pigment emission

- Adjust the amount of pigment emission by turning the adjustment ring.



### Strong packaging of needle cartridges

- Packaging with transparent materials
- Prevent damage to products by using strong packaging



### Product information

# DIGITAL-POP DELUXE

**Product name :** Semi-permanent make-up machine

**Model name :** DIGITAL-POP DELUXE



This product is 'Medical device', please be sure to read the instructions before use and follow the user manual. Please contact us at the contact number below if you have any inquiries about the product.

## Composition and Introduction

### Composition

Body + Hand-piece + Foot switch + Adapter + Cable

### Introduction

This device is used for injecting pigmentation inhaled into the disposable needle to mark the color pigment to skin, or used for irritating skin to help medical supplies to be absorbed well.



10 speed level



Generating Shortcut key for saving speed



Turn ON/OFF power with 'Foot switch' without controlling main body.



Dual functions of MTS/PMU



## Easy to use

- Easy to adjust the speed by clicking a UP/DOWN button.
- Visualized current speed on LED display.
- Speed Level Save and operate at the saved speed.



## Convenient power ON/OFF


- Convenient for users to turn ON/OFF power with 'foot switch' without controlling the main body.
- Long press for power on/off, Short press for Pause.
- Motor stops when you place a hand-piece on a stand



Disposable needle cartridge



- Selectable disposable needle cartridge between PMU and MTS for its purpose.
- Adjust the speed and power of hand-piece by clicking the main button.
- Operate with various types of needle.

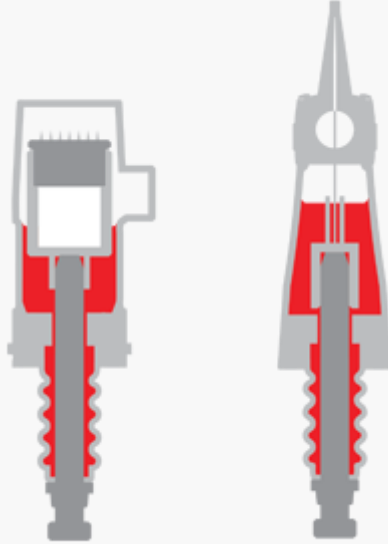


### Disposable sterilized cartridges

- EO sterilized disposable needle cartridges  
(Purchase separately, consumables)



### Applied our own patent technology

- Hygienically prevents from being injected contaminant during the operation.
- Adjust the amount of pigment by turning a ring
- ※ Needle Cartridges are purchasable separately

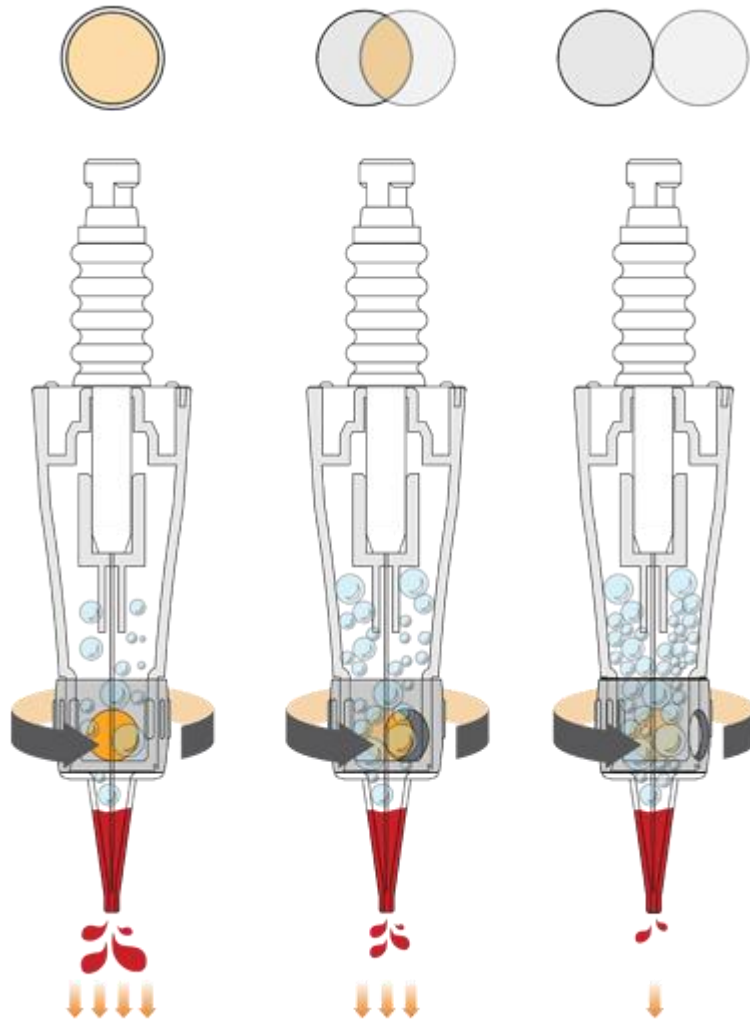

### Adjust the amount of pigment emission

- Adjust the amount of pigment emission by turning the adjustment ring.



### Strong packaging of needle cartridges

- Packaging with transparent materials
- Prevent damage to products by using strong packaging



# DIGITAL HAND-R

**Product name** : Semi-permanent make-up machine

**Model name** : DIGITAL HAND-R



This product is 'Medical device', please be sure to read the instructions before use and follow the user manual.  
Please contact us at the contact number below if you have any inquiries about the product.

## Composition and Introduction

### Composition

Body + Hand-piece stand + Adapter

### Introduction

This device is used for injecting pigmentation inhaled into the disposable needle to mark the color pigment to skin, or used for irritating skin to help medical supplies to be absorbed well.



7 speed level



Pause on the stand



Dual functions of MTS/PMU




## Easy to use

- Easy to adjust the speed level from 1 to 7 by pressing a button.
- Visualized current speed on LED display of the hand-piece body.
- Selectable disposable needle cartridge between PMU and MTS for its purpose.



### Pause function for user's convenience


- User can make hand-piece stop by putting it down on the stand.
- Convenient pause without any button controls.

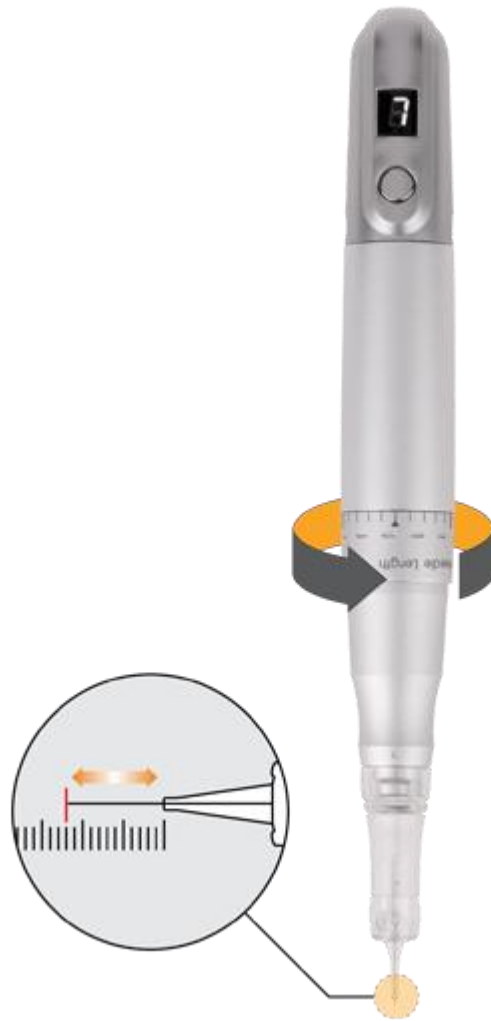



## Streamlined design of Hand-piece

- Comfortable design to hold the hand-piece for a long time.
- Operate with various types of needle.
- Easy to adjust the needle length by turning the adjustment ring.

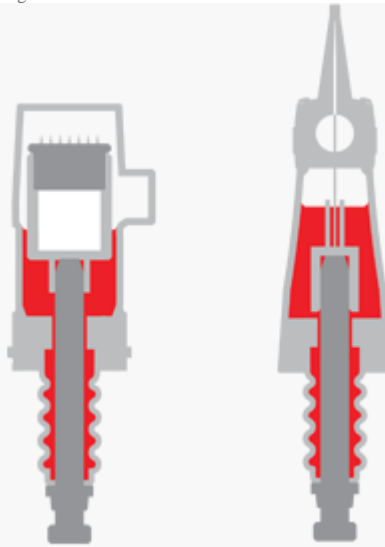


## Disposable sterilized cartridges

- EO gas Sterilized disposable needle cartridges



## Applied our own patented technology

- Prevent backflow of blood or infection during operation.
- ※ Purchase separately and single use

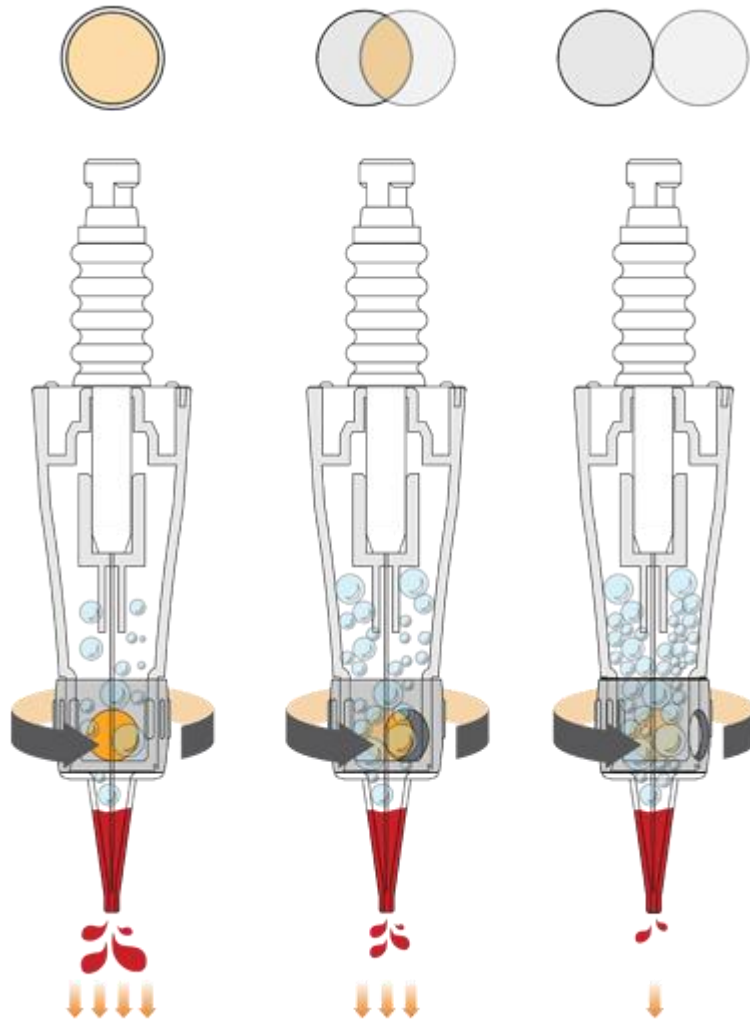

### Adjust the amount of pigment emission

- Adjust the amount of pigment emission by turning a cartridge's ring.
- ▶ If needle cartridge's hole is closed, little pigment is emitted. If needle cartridge's hole is open, much pigment is emitted.





### Strong packaging of needle cartridges

- Packaing with transparent materials
- Prevent damage to products by using strong packaging.
- ※ Purchase separately and single use



# DIGITAL PEN-A

**Product name** : Semi-permanent make-up machine

**Model name** : DIGITAL PEN-A



This product is 'Medical device', please be sure to read the instructions before use and follow the user manual.  
Please contact us at the contact number below if you have any inquiries about the product.

## Composition and Introduction

### Composition

Body + Hand-piece stand + Adapter

### Introduction

This device is used for injecting pigmentation inhaled into the disposable needle to mark the color pigment to skin, or used for irritating skin to help medical supplies to be absorbed well.



5 speed level



Pause on the stand



Dual functions of MTS/PMU



## Easy to use

- Easy to adjust the speed level from 1 to 5 by pressing a button.
- Button flickers according to the current speed
- Selectable disposable needle cartridge between PMU and MTS for its purpose.




### **Pause function for user's convenience**

- User can make hand-piece stop by putting it down on the stand.
- Convenient pause without any button controls.



### **Streamlined design of Hand-piece**

- Comfortable design to hold the hand-piece for a long time.
- Operate with various types of needle.
- Easy to adjust the needle length by turning the adjustment ring.

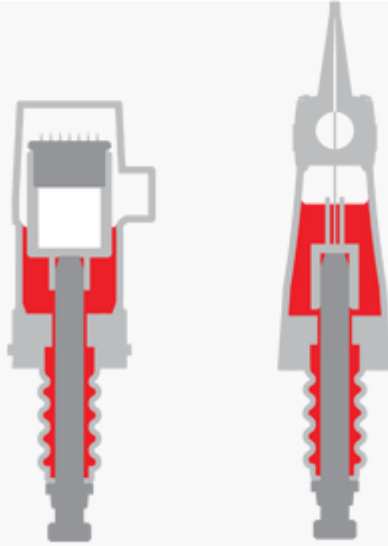


### Disposable sterilized cartridges

- EO gas Sterilized disposable needle cartridges



### Applied our own patented technology

- Prevent backflow of blood or infection during operation.
- ※ Purchase separately and single use

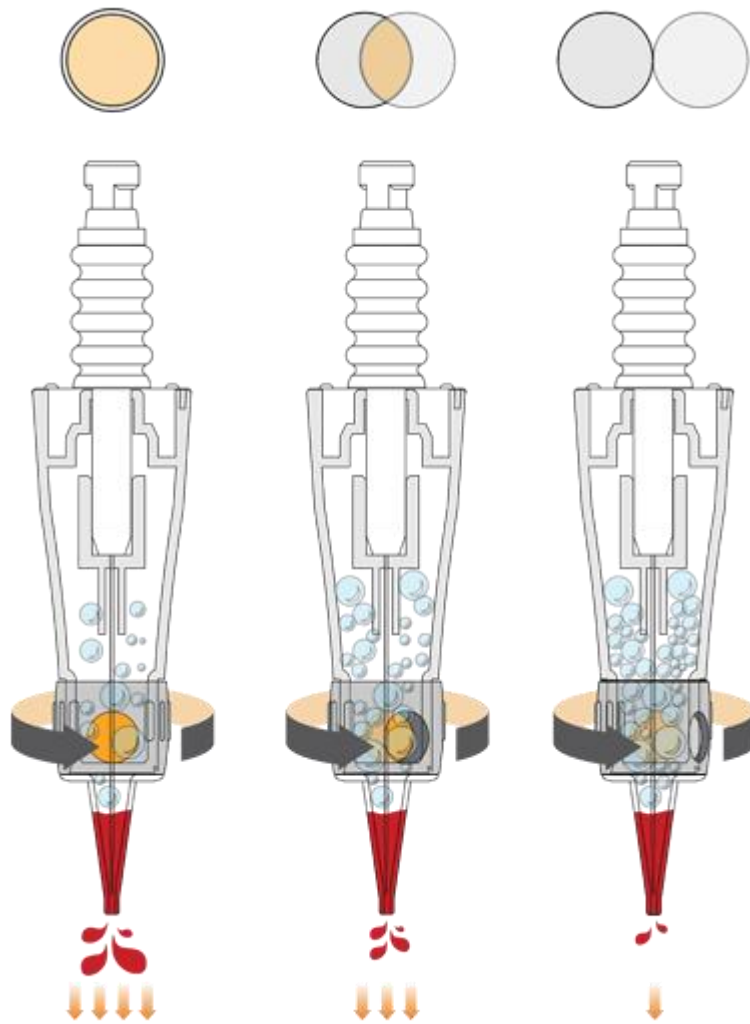


### Adjust the amount of pigment emission

- Adjust the amount of pigment emission by turning a cartridge's ring.
- ▶ If needle cartridge's hole is closed, little pigment is emitted. If needle cartridge's hole is open, much pigment is emitted.



### Strong packaging of needle cartridges

- Packing with transparent materials
- Prevent damage to products by using strong packaging.
- ※ Purchase separately and single use

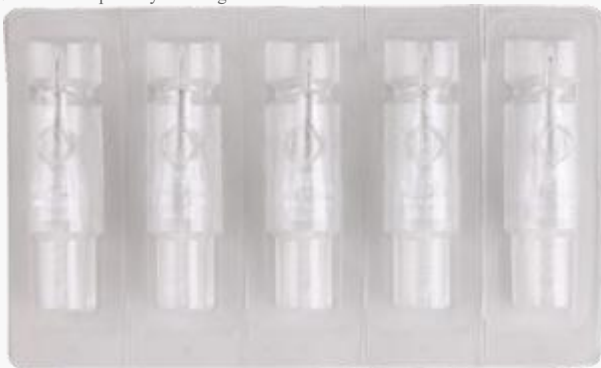



# DIGITAL-POP

**Product name** : Semi-permanent make-up machine

**Model name** : DIGITAL-POP



This product is 'Medical device', please be sure to read the instructions before use and follow the user manual. Please contact us at the contact number below if you have any inquiries about the product.

## Composition and Introduction

### Composition

Body + Hand-piece + Foot switch + Adapter + Cable

# Introduction

It marks the color pigments inhaled into the needle by injecting them to skin by a back-and-forth motion.

MTS machine helps medical supplies be absorbed to skin by irritating.



10 speed level



Generating Shortcut key for saving speed



Power ON/OFF by foot switch



Dual functions of MTS/PMU



Easy to use

- Easy to adjust the speed by clicking a UP/DOWN button.
- Long press for power on/off, Short press for Pause.
- Visualized current speed on LED display.
- Speed Level Save and operate at the saved speed.



## Convenient power

### ON/OFF


- Convenient for users to turn ON/OFF power with 'foot switch' without controlling the main body.





## Disposable needle cartridge

- Selectable disposable needle cartridge between PMU and MTS for its purpose.
- Adjust the speed and power of hand-piece by clicking the main button.
- Operate with various types of needle.

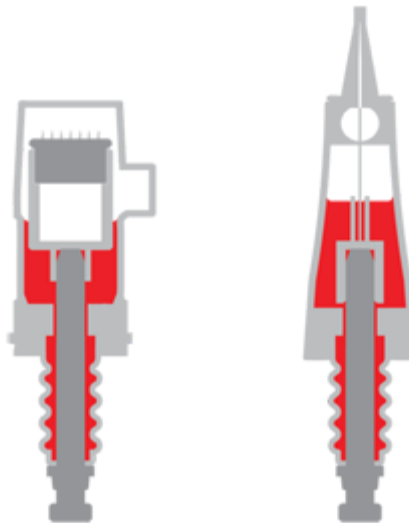

## Disposable sterilized cartridges

- Sterilized disposable needle cartridges (Purchase separately, consumables)



## Applied our own patent technology

- Hygienically prevents from being injected contaminant during the operation.
- ※ Needle Cartridges are purchasable separately

# DIGITAL-HAND-WF

**Product name** : Semi-permanent make-up machine

**Model name** : DIGITAL-HAND-WF



This product is 'Medical device', please be sure to read the instructions before use and follow the user manual.  
Please contact us at the contact number below if you have any inquiries about the product.

# Composition and Introduction

## Composition

Body + Battery + Adapter + Stand + Charger + DC Input Cab

## Introduction

It marks the color pigments inhaled into the needle by injecting them to skin by a back-and-forth motion.

MTS machine helps medical supplies be absorbed to skin by irritating.



5 speed level



Both wired and wireless



Pause on the stand



Dual functions of MTS/PMU



## Easy to use

- Adjust 5 steps of speed and power by clicking the button.
- Easy to adjust the length of needle by turning the adjustment ring. (Adjustable 0 to 2mm for MTS, 1.5 to 4mm for PMU)
- Change needles between MTS/PMU for its purpose.
- Simple operation



Round



Flat



## grasp feelings

- Comfortable hand-piece design for users
- Light weight





## **Pause for user's convenience.**

- Pause right after putting down the hand-piece on the stand.
- Convenient pause without any button controls.



## Equipped chargeable battery


- Wireless use with the chargeable battery





## Both Wired and Wireless

- Optional use between Wired and Wireless on user's preference and the circumstances.

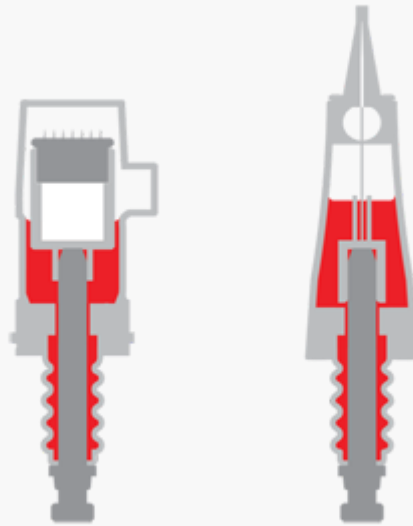

## Disposable sterilized cartridges

- Sterilized disposable needle cartridges (Purchase separately, consumables)



## Applied our own patent technology

- Hygienically prevents from being injected contaminant during the operation.
- ※ Needle Cartridges are purchasable separately

